



# NON-REBREATHING OXYGEN MASK

## USER MANUAL



## Contents

---

Intended Use	1
Product Overview	2
Specification	3
Operation Instruction	3
Cleaning and Disinfection	3
Warranty	4
Certificate	4
Warning	4

## Intended Use


---

Oxygen Mask is designed for patient comfort with a soft and anatomical form.

This disposable non-rebreathing oxygen mask has an attached 1-liter reservoir bag that connects to an external oxygen tank or bulk oxygen supply system. It is designed for patients requiring high concentrations of oxygen. The elastic head strap keeps the mask in place and the soft, clear finish helps with greater patient comfort and easier monitoring of the patient condition.

## Product Overview

---

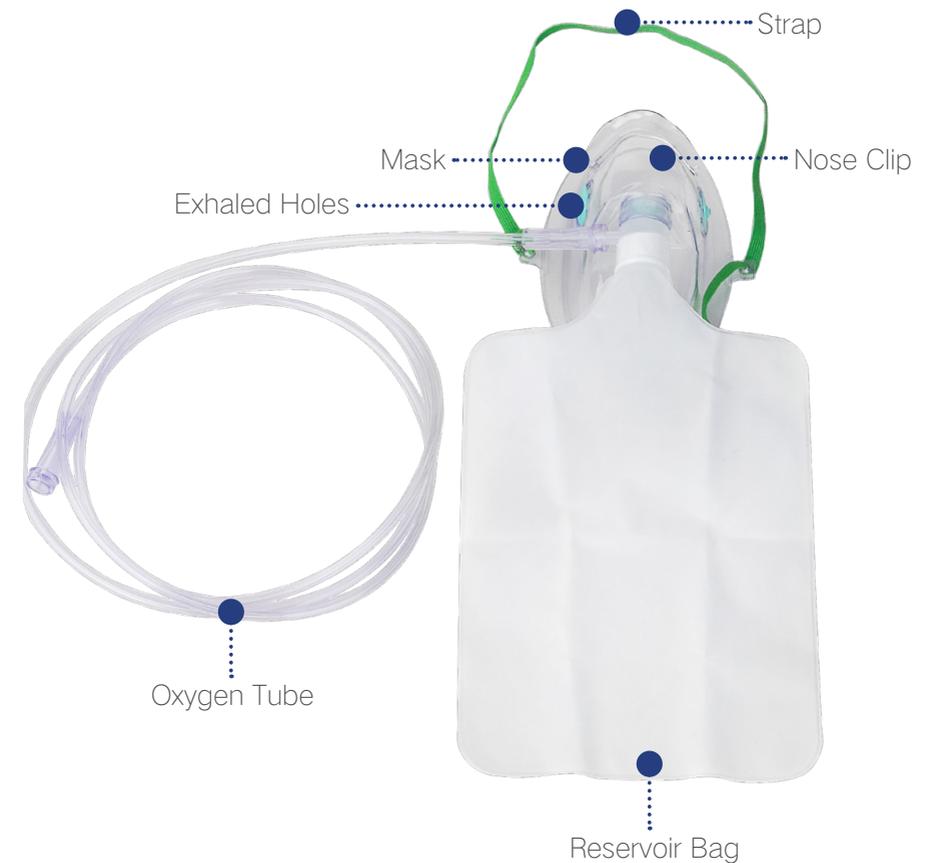

## Specification

---

Item No.	Size	Quantity/Carton	Shipping Weight
OT-DOM-T02-TCT-4	Adult-L	50	5.0kg
OT-DOM-T02-TCT-1	Pediatric	50	4.6kg

The non-rebreather mask is able to provide high oxygen concentration ranges from 60% to 80%.

## Operation Instruction

---

- ① Remove plastic packaging from mask kit.
- ② Fix the reservoir bag and oxygen tube to the mask.
- ③ Put the mask correctly and softly to the patient's mouth in one hand, expand the elastic head strap with the other hand just enough to fit comfortably over patient's head.
- ④ Connect the oxygen tubing to exterior oxygen source or device and set the suitable flow rate.

## Cleaning and Disinfection

---

The Non-rebreathing Oxygen Masks are designed for single use only. Do not re-use. Re-use will lead to increased risk of cross contamination, degradation of performance and/or device malfunction.


## Warning

---

- ① Use of non-FirstCare parts may affect safety and/or performance.
- ② Re-use can result in cross-infection. Do not soak, rinse, wash, autoclave this product. These procedures may leave harmful residues on the mask.
- ③ FirstCare is not responsible for any consequences of re-use of the disposable Non-rebreathing Oxygen Mask.

## Certificate

---

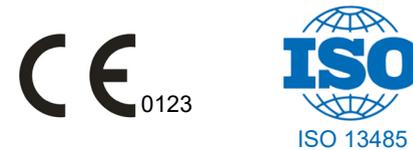


## Warranty

---

This equipment is manufactured from the finest quality materials. Each individual part is subject to strict quality control tests to ensure exceptionally high standards.

The manufacturer warrants to the purchaser of FirstCare Non-rebreathing Oxygen Mask that its component parts are free from defects in material and workmanship for a period of one year from the date of purchase. The manufacturer will replace and/or repair all parts of the mask at its option for one year from the date of purchase at no cost to the purchaser, upon the notification of the defects, in writing by the purchaser. All shipping costs shall be borne by the purchaser.

The manufacturer shall be liable under this warranty only if the masks have been used in the normal manner described in the instruction manual.

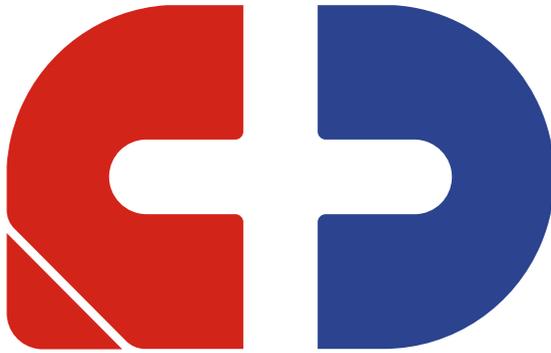

**FIRSTCARE™**  
WE EQUIP REAL-LIFE HEROES

Xiamen FirstCare Imp. & Exp. Co.,Ltd.

📍 1803, BLDG. #1-T3, SEC Taidi Haixi  
Center Xiamen, China 361026

🌐 [www.firstcaresolutions.com](http://www.firstcaresolutions.com)

☎ 0086-592-6531682

✉ [info@firstcaresolutions.com](mailto:info@firstcaresolutions.com)

Connect with us:

