



# SIMPLE OXYGEN MASK

## USER MANUAL



## Contents

---

Intended Use	1
Product Overview	2
Specification	3
Operation Instruction	3
Cleaning and Disinfection	3
Warning	4
Certificate	4
Warranty	5

## Intended Use

---


Oxygen Mask is designed for patient comfort with a soft and anatomical form.

This disposable oxygen mask is our most fundamental oxygen conveyor with 40-50% oxygen concentration. It is made from medical grade PVC material and the whole set consists of mask, oxygen tube, elastic trap and connector (corrugated tube selectable). This O2 mask is latex-free with smooth edges that are designed for patient comfort while reducing irritation points.

A rotating joint connected between the mask and the oxygen tube is selectable for customers to help adjusting the oxygen tube once it get twisted.

## Product Overview

---

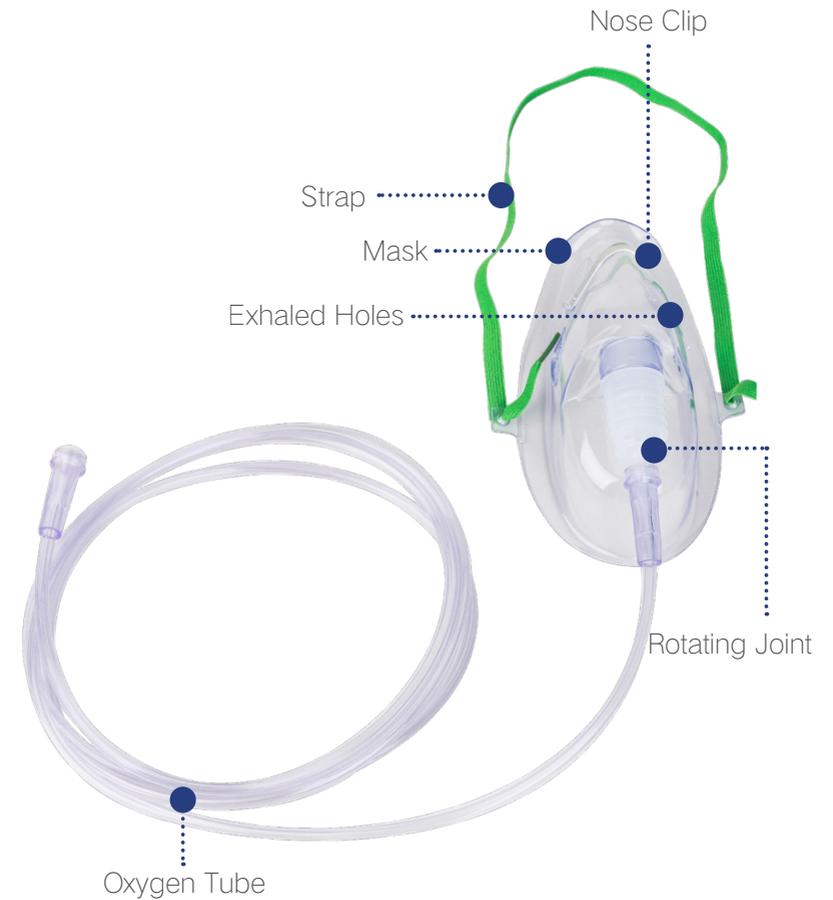

## Specification

---

P/N	Rotating Joint	Size	Quantity/ carton	Size/carton	Weight/ Carton
OT-DOM-T01-4	Without	Adult-L	50	46*36*30cm	4.7kg
OT-DOM-T01-1	Without	Pediatric	50	46*36*30cm	4.1kg
OT-DOM-T01-TRJ-4	With	Adult-L	50	46*36*30cm	4.7kg
OT-DOM-T01-TRJ-1	With	Pediatric	50	46*36*30cm	4.1kg

## Operation Instruction

---

- 1 Remove plastic packaging from mask kit.
- 2 Fix the oxygen tube to mask.
- 3 Put the mask correctly and softly to the patient's mouth in one hand, expand the elastic head strap with the other hand just enough to fit comfortably over patient's head.
- 4 Connect the oxygen tubing to exterior oxygen source or device and set the suitable flow rate.

## Cleaning and Disinfection

---

The PVC Oxygen Masks are designed for single use only. Do not re-use. Re-use will lead to increased risk of cross contamination, degradation of performance and/or device malfunction.


## Warning

---

- 1 Use of non-FirstCare parts may affect safety and/or performance.
- 2 Re-use can result in cross-infection. Do not soak, rinse, wash, autoclave this product. These procedures may leave harmful residues on the mask.
- 3 FirstCare is not responsible for any consequences of re-use of the disposable PVC Oxygen Mask.

## Certificate

---

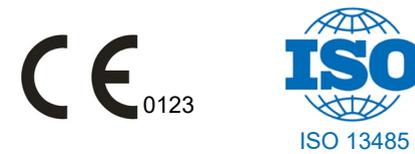

## Warranty

---

This equipment is manufactured from the finest quality materials. Each individual part is subject to strict quality control tests to ensure exceptionally high standards.

The manufacturer warrants to the purchaser of FirstCare PVC Oxygen Mask that its component parts are free from defects in material and workmanship for a period of one year from the date of purchase. The manufacturer will replace and/or repair all parts of the mask at its option for one year from the date of purchase at no cost to the purchaser, upon the notification of the defects, in writing by the purchaser. All shipping costs shall be borne by the purchaser.

The manufacturer shall be liable under this warranty only if the masks have been used in the normal manner described in the instruction manual.



XIAMEN FIRSTCARE IMP. & EXP. CO., LTD.

1803, BLDG #1-T3, SEC Taidi Haixi Center, XIAMEN, CHINA, 361026

Email : [info@firstcaresolutions.com](mailto:info@firstcaresolutions.com)

Tel : +86-592-6531682

[www.firstcaresolutions.com](http://www.firstcaresolutions.com)